



Bluetooth Connection Guide

for T4B Mobility Scooters 2026



VOLUME UP



VOLUME DOWN



PREVIOUS SONG



FORWARD SONG



BLUETOOTH ON/OFF



USB-A CHARGER

Flip cover for access



HOW TO CONNECT

- 1 Turn on the Bluetooth by pressing the Bluetooth ON/OFF button.
- 2 Activate Bluetooth on your phone/device.
- 3 From the list of available devices, select and pair with:
“053..”
- 4 Once connected, the scooter's speaker will be connected to your device.



NOTE: If you experience any connection issues, turn off the Bluetooth, wait a few seconds, and try pairing again.